

Washing Instructions:

- Wash your compression stockings separately or with garments of the same colour in the washing machine at a temperature of 30 or 40 degrees.
- Please wash them on a gentle wash cycle.
- Use a simple colour detergent without brighteners. Never use fabric softeners.

Fitting instructions:

- Put your toes into the sock and bring it over your heel before sliding it up your leg; to remove, pull the top of the sock down,
- Caution- if pain and discomfort occur while wearing, remove and consult your physician



temperature



no bleach




low heat tumble



no ironing



no dry clean

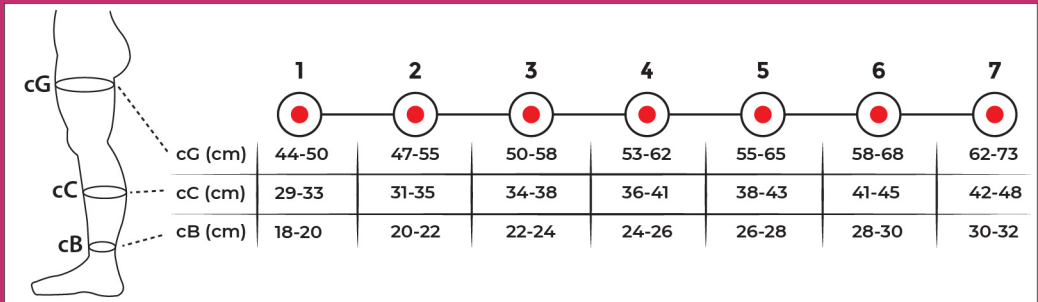

PV-100



PV-101



PV-104



PV-114